

---

The date of issue: 2025-07-29

Revision: 002

# Transparent Dressing

## Instruction for use

**Description:** Transparent Dressing is breathable, allowing good oxygen and moisture vapor exchange. It is water resistant.

According to the different structural composition, Transparent Dressing is divided into standard type and plus pad type. Standard Type consist of film substrate with non-latex, hypoallergenic adhesive. Plus Pad Type consist of absorbent pad, film substrate with non-latex, hypoallergenic adhesives.

**Intended use:** The transparent dressing can be used in conjunction with other instruments to provide fixation, or as a mechanical barrier, for compression or for absorption of exudates.

Common applications include:

- Catheter sites
- IV sites
- Surgical incisions
- Minor burns
- Donor sites
- Secondary dressing
- Superficial wounds such as abrasions and skin ulcers

**Contraindications:** Not intended for use on third degree burns.

**Warnings:** Do not use the dressing as a replacement for sutures and other primary wound closure methods.

### Precautions:

- Stop any bleeding at the site before applying dressing.
- Do not stretch the dressing during application as tension can cause trauma to the skin.
- Do not use if package is damaged or opened prior to use.

- 
- Single-use only.
  - To avoid issues with cross-contamination and sterility, use immediately and discard any unused dressing, the dressing should not be re-used once package is opened.
  - Make sure the skin is clean and free of soap residue and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
  - Transparent dressing should not be re-sterilized.
  - The dressing may be used on an infected site, only when under the care of health care professional.
  - Do not use on patients with known sensitivity to the dressing or its component.

**Direction for use**

**Dressing selection:** Choose a dressing that is a size larger than the wound or catheter site in order to provide a one inch margin for the film to adhere to dry, health skin.

**Site preparation:**

1. Prepare the site according to institution protocol.
2. Removal of excess hair may be necessary to ensure proper adhesion.
3. Allow the area and surrounding areas to dry completely.

**Application:**

1. Open the package and remove the sterile dressing.
2. Peel the release liner from the dressing exposing the adhesive surface.
3. Center the dressing over the site or wounds.
4. Press the dressing into place.
5. Slowly remove the face liner by lifting the pull tab while smoothing down the dressing edges.
6. Smooth the dressing from the center, using firm pressure to enhance adhesion.
7. For dressing supplied with a label, record information and then place it on or near the dressing.

**Site care:**

1. The site should be observed for signs of infection or other complications. If infection is suspected, remove the dressing, inspect the site directly and determine

---

appropriate medical intervention.

2. Change the dressing according to institution protocol or when the barrier properties have been compromised.

**Removal:**

Gently grasp an edge and slowly peel the dressing from the skin in the direction of hair growth. Avoid trauma by peeling the dressing back, rather than pulling it up from the skin. Alternatively, for removal from IV sites or other devices, grasp one edge of the dressing and gently pull it straight out to stretch it and release adhesion. Care should be taken not to dislodge catheters or other devices or to disrupt the wound surface when the dressing is removed. Support the skin and the catheter while removing the dressing. If the dressing adheres to the wound surface, it should be gently soaked off.

**Note:** to aid in lifting the dressing edge, secure a piece of surgical tape to one corner and rub firmly. Use tape as a tab to help slowly peel back the dressing.

**Shelf life:**

5 years

**Sterilization:**


Sterilized using ethylene oxide

**Storage:**

store in room temperature, keep dry, and avoid directly sunlight.

**Symbol Instruction**

|                                                                                     |                     |                                                                                     |                          |
|-------------------------------------------------------------------------------------|---------------------|-------------------------------------------------------------------------------------|--------------------------|
|  | Medical device      |  | Unique device identifier |
|  | Manufacturer        |  | Caution                  |
|  | Date of manufacture |  | Use-by date              |
|  | Batch code          |  | CE mark                  |

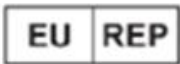
|                                                                                    |                                                                    |                                                                                   |                                                                 |
|------------------------------------------------------------------------------------|--------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------|
|   | Authorized representative in the European Community/European Union |  | Single sterile barrier system with protective packaging outside |
|   | Sterilized using ethylene oxide                                    |  | Do not use if package is damaged                                |
|   | Single sterile barrier system                                      |  | Do not re-use                                                   |
|   | Catalogue number                                                   |  | Consult instructions for use                                    |
|   | Keep dry                                                           |  | Fear of the sun                                                 |
|  | Do not re-sterilize                                                |                                                                                   |                                                                 |



**Name:** Jiangsu NewValue Medical Products Co.,Ltd.

**Address:** No.22 Zelan Road, China Medical City, Taizhou 225300 Jiangsu,P.R.China.

**SRN:** CN-MF-000008202

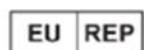

**Name:** SUNGO Europe B.V.

**Address:** Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

**Website Address:** [www.sungoglobal.com](http://www.sungoglobal.com)

**Contact Person incl. Function:** Yana Zhang (Ms.)

**Contact Phone:** +31(0) 10 3034500

**Contact E-Mail:** [ec.rep@sungogroup.com](mailto:ec.rep@sungogroup.com)/[yan.zhang@sungoglobal.com](mailto:yan.zhang@sungoglobal.com)

**SRN:** NL-AR-000000247