

Hydrocolloid Dressing

Instruction for Use

Description:

Hydrocolloid Dressing is sterile wound dressing consisting of an inner layer of hydrocolloid contained with an adhesive polymer matrix and outer layer of polyurethane film.

Over the wound site, Hydrocolloid Dressing interact with wound fluid to create a soft semi-transparent absorber mass, which can maintain a moist environment to protect wound and enhance wound healing.

Intended Use

Hydrocolloid Dressing is intended to be used principally with wounds which have breached the dermis and can only heal by secondary intent.

Indications

Chronic Wounds – leg ulcers, superficial pressure sores.

Acute Wounds - Surgical wounds (post-operative, donor site, dermatological excisions), Burns (first and second degree), Minor abrasions and minor lacerations

May be used as a secondary dressing in combination with other modern wound care products.

Patient populations

For patients of all ages who require wound care.

Intended user

Medical staff and patients requiring wound care.

Contraindications:

Should not be used on third degree burns and on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components.

Precautions

- Hydrocolloid dressing is for external use only.
- Do not use if package is damaged or opened prior to use.
- Single-use only.
- To avoid issues with cross-contamination and sterility, use immediately and discard any unused dressing, the dressing should not be re-used once package is opened.
- Do not use the product in combination with other wound care products without first consulting a healthcare professional.

- Unnecessary material is removed from the wound during the body's normal healing process, which could make the wound appear larger. If the wound continues to get larger after the first few dressing changes, consult a healthcare professional.
- If irritation or maceration occurs, discontinue use and review the dressing protocol.
- Do not use on patients with a known sensitivity to hydrocolloid dressing or its component.
- Do not use on patients who have third degree burns.
- Treatment of leg ulcers, diabetic ulcers, pressure ulcers, burns (first and second degree), surgical wounds and traumatic wounds should be under the supervision of a healthcare professional.

Directions for Use

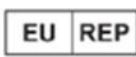
- Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.
- Apply the dressing to the wound. Hydrocolloid Dressing should extend at least 1/4 inch (3.2cm) beyond the edges of the wound.
- Remove the release paper from the back of the dressing being careful to minimize finger contact with the adhesive surface.
- Hold the dressing over the wound and line up the center of the dressing with the center of the wound.
- Gently roll the dressing into place over the wound.
- All wounds should be inspected frequently. Remove Hydrocolloid Dressing when exudate fully saturates the dressing.
- The single Hydrocolloid Dressing may be remained on a wound up to 7 days. Cumulative single use or contact not exceeds 28 days.

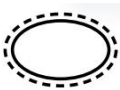
Storage:

1. Keep dry
2. Keep away from sunlight directly

Shelf Life: 3 years

Index of CE symbol:

	Manufacturer		Date of manufacture
	Authorized representative in the European Community		Use-by date
	Catalogue number		Batch code
	Sterilized using irradiation		Do not resterilize

	Consult instructions for use or consult electronic instructions for use		Do not re-use
	Caution		Do not use if package is damaged and consult instructions for use
	Keep away from sunlight	CE 0123	CE Mark & Notified Body
	Keep dry	MD	MEDICAL DEVICE
UDI	Unique Device Identifier		Single sterile barrier system
	Single sterile barrier system with protective packaging outside		



Name: Jiangsu NewValue Medical Products Co., Ltd.

Add: No.22 Zelan Road, China Medical City, Taizhou 225300 Jiangsu, P.R.China

SRN: CN-MF-000008202

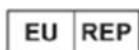
Name: SUNGO Europe B.V.

Add: Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

SRN: NL-AR-000000247

Tel/Fax: +31(0)10 3034500

E-mail: ec.rep@sungogroup.com/yan.zhang@sungoglobal.com



Size

Type	Length(cm)	Width(cm)
Standard	2-20	1.5-20
Bordered	2-20	1.5-20
Island	2-20	1.5-20