

Silicone Clear Dressing with Hydrogel

File No: JSNWL-MDR-TCF-03-2.1

Prepared/date: 2025.07.29

Issue date:2025.07.29

Product Appearance and Structure

The Silicone Clear Dressing with Hydrogel consists of PU film, hydrogel, silicone adhesive and a protective layer.

Product description

The Silicone Clear Dressing with Hydrogel consists of PU film, hydrogel, silicone adhesive and a protective layer. The protective layer will be removed before use. The Silicone Clear Dressing with Hydrogel helps redistribute pressure and manage microclimate. The Silicone Clear Dressing with Hydrogel provides visibility to the skin without lifting the dressing. The hydrogel draws heat away from skin creating a cooling effect. The silicone face and border aids of the Silicone Clear Dressing with Hydrogel can be gently removed.

Intended use

Silicone Clear Dressing with Hydrogel is used as part of a pressure injury prevention protocol and is ideal for partial and full-thickness wounds with low exudate.

Contraindications

Third degree burns.

Do not use on patients with a known sensitivity to any of the product components.

Side-effects

In rare cases Silicone Clear Dressing with Hydrogel might cause local skin reactions (erythema, maceration).

Warnings and precautions

Silicone Clear Dressing with Hydrogel is packaged for single use. Do not re-use or re-sterilize as this is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Discard open or unused material of the device. The wound should be inspected for signs of infection and -in case- treated according to clinical practice.

Indications

It can be used for the care of non-chronic wounds (such as superficial wounds, sutured wounds after surgery, mechanical wounds, small wounds, abrasions, cut wounds, puncture sites of puncture equipment, I or shallow II degree burn wounds)

Intended Patient Population

For patients who require wound care.

Intended User

Intended use for health care professionals or lay-persons.

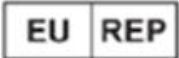
Shelf Life

3 years

Directions for use

- Do not cut Silicone Clear Dressing with Hydrogel.
- Choose a dressing size that covers the wound or gluteal fold.
- Open the pouch and take out Silicone Clear Dressing with Hydrogel. Ensure sterility.
- Gently lift one corner, then slowly lift off the dressing.
- Remove release liner.
- Gently secure the dressing to the skin.
- Frequency of dressing change depends on the condition of the wound.
- Clean the wound according to good clinical practice prior to application of a new dressing.
- Silicone Clear Dressing with Hydrogel can be used exceed 24 h but not exceed 28 days depending on the amount of drainage/moisture.

Symbol instruction

	Manufacturer		Caution
	Date of manufacture		Use-by date
	Batch code		Catalogue number
	Authorized representative in the European Union		Do not resterilize
	Do not use if package is damaged		Sterilized using ethylene oxide
	Keep dry		Unique device identifier
	Medical device		Do not re-use
	Keep away from sunlight		“CE” Marking
	Consult instructions for use		Single sterile barrier system

	Single sterile barrier system with protective packaging outside		
---	--	--	--

Storage

Protect from freezing, avoid excessive heat.



Jiangsu NewValue Medical Products Co.,Ltd.

No.22 Zelan Road, China Medical City, Taizhou, 225300 Jiangsu, China.

SRN: CN-MF-000008202

Contact phone: 0523-86867878



SUNGO Europe B.V.

Add: Fascinatio Boulevard 522, Unit 1.7 , 2909VA Capelle aan den IJssel,
The Netherlands

Contact Phone: +31(0)10 3034500

SRN: NL-AR-000000247

E-mail: ec.rep@sungogroup.com/yan.zhang@sungoglobal.com