

Instructions for Use

Document No.: JSNWL-MDR-TCF-06-2.1

Version: 001

Issue date: 2025-07-29

Device name: Skin closure Strips

Product description

The standard type of Skin closure Strips is a tape consisting of Non-woven Fabric and acrylic adhesive.

The elastic type of Skin closure Strips is a tape consisting of elastic Non-woven Fabric and acrylic adhesive

The reinforced type of Skin closure Strips is a tape consisting of Non-woven Fabric and polyester fiber and acrylic adhesive.

Indications

Skin closure Strips are an effective treatment for closing securing and supporting small cuts and minor wounds. They help to keep the edges of wounds together to minimize the opportunity for re-opening resulting in future bleeding, scab formation and scarring.

Contraindications

Do not use on patients with a known sensitivity to any of the product components.

Warnings and precautions

Skin closure Strips is packaged for single use. Do not re-use or re-sterilize as this is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Discard open or unused material of the device. The wound should be inspected for signs of infection and -in case- treated according to clinical practice.

Directions for use

- Clean and dry the affected area.
- Remove the closure from the pouch.
- Tear away the small perforated tab from one end of the backing card.
- Apply half of the closure to the edge of the wound.
- Draw together the edges of the wound and apply the closure fully across the wound pressing firmly into place .
- Place additional strips across the wound as required.
- For ease of removal peel back the strip from each end towards the cut.
- Gently remove the whole strip.

Storage

Skin closure Strips should be stored in dry conditions. Do not expose to direct sunlight.


Sterilization:

Sterilized using ethylene oxide

Shelf Life:

5 years

Symbol Instruction

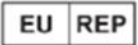
	Medical device		Unique device identifier
	Manufacturer		Caution
	Date of manufacture		Use-by date
	Authorized representative in the European Community/European Union		Single sterile barrier system with protective packaging outside
	Batch code		Catalogue number
	Sterilized using ethylene oxide		Do not use if package is damaged
	Single sterile barrier system		Do not re-use
	Keep dry		Fear of the sun
	Consult instructions for use		CE mark
	Do not re-sterilize		



Jiangsu NewValue Medical Products Co.,Ltd.

Address: No.22 Zelan Road, China Medical City, Taizhou 225300 Jiangsu,P.R.China.

SRN: CN-MF-000008202

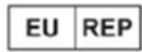

Name: SUNGO Europe B.V.

Address: Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The
Netherlands

Contact : Yana Zhang (Ms.)

Tel/Fax: +31(0) 10 3034500

E-Mail: ec.rep@sungogroup.com/yan.zhang@sungoglobal.com

SRN: NL-AR-000000247